




medw^ork®

HOT

Series

Hot Biopsy Forceps

HOT Series: Hot Biopsy Forceps

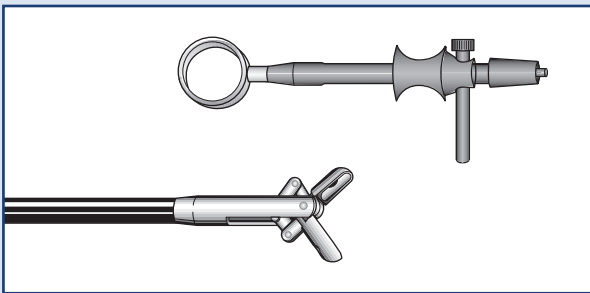

The hot biopsy forceps in the HOT series are used to excise sessile polyps in the upper and lower gastrointestinal tract using high-frequency coagulation current. The hot biopsy forceps are reusable and autoclavable.

To ensure optimum protection for patients, staff and others, we advise you to observe the reprocessing guide delivered with the product.

medw^{ork}®

TIP!

To prevent perforation of the intestinal wall the polyp should be pulled into the intestinal lumen with the hot biopsy forceps when coagulation current is applied. The hot biopsy forceps are suitable for the excision of polyps up to 8 mm in size. Larger polyps should be excised with a diathermia snare.

The service life of hot biopsy forceps depends on various factors and is prolonged considerably if the following advice is heeded:

- ▶ Only advance the forceps in small steps in the working channel of the endoscope.
- ▶ The handle of the forceps should always match the endoscope channel.
- ▶ Avoid loops and flexions.
- ▶ Only work with light pressure when closing the cups.
- ▶ After use, clean and sterilise the forceps every time.



Please pay attention to the instruction manual!

The service life of reusable devices is dependent on care, cleaning, sterilisation and disinfection, as well as on the number of uses.

■ Product Code System: HOT2-E1-25-230 (example)

Product Group	Product Category	Shape / Function	Material / Characteristics	Technical Data 1	Technical Data 2
HOT2-E1-25-230	HOT2-E1-25-230	HOT2-E1-25-230	HOT2-E1-25-230	HOT2-E1-25-230	HOT2-E1-25-230
▼	▼	▼	▼	▼	▼
HOT = Hot biopsy forceps	2 = Reusable hot biopsy forceps	E = Long oval spoons	1 = Isolated	ø Tube 18 = 1,8 mm 23 = 2,3 mm 25 = 2,5 mm	Length 160 = 160 cm 230 = 230 cm 260 = 260 cm

Technical Data

HOT2 – Reusable hot biopsy forceps

Product Code Order Number	Shape	Characteristics	ø Tube	Length	
HOT2-E1-23-160 500675	Long oval spoons	Fenestrated, isolated	2,3 mm	160 cm	
HOT2-E1-23-230 500676	Long oval spoons	Fenestrated, isolated	2,3 mm	230 cm	
HOT2-E1-25-160 500678	Long oval spoons	Fenestrated, isolated	2,5 mm	160 cm	
HOT2-E1-25-230 500679	Long oval spoons	Fenestrated, isolated	2,5 mm	230 cm	

Accessories

HAN2 – Reusable handles and adapter

Product Code Order Number	Characteristics	Lift	HF power connector	
HAN2-050-OL 500650	Rotatable	50 mm	Olympus	
HAN2-050-ST 500651	Rotatable	50 mm	Standard	
HAN2-Adapter-OL-ST 500654	Converts Olympus- into standard power connection			

MIS2 – Reusable High-Frequency-Cable

Product Code Order Number	Plug	HF power connector	
MIS2-HF-05-OL 500962	On the part of appliance 5 mm	Olympus	
MIS2-HF-05-ST 500963	On the part of appliance 5 mm	Standard	

Special models available on demand. Errors excepted. Technical data are subject to change without notice.